

SurgiVet

Clinical Support Services

Toll-Free USA: 888-745-6562

Recovery O2 Mask for Supplemental Oxygen Delivery

The Recovery O2 Masks can be used to provide oxygen to animals that are in respiratory distress. The mask has a 22mm x hose barb adapter on the inlet side, allowing you to connect either oxygen tubing (included) or the adapter can be removed to allow the attachment of a standard 22mm anesthesia machine breathing circuit. The mask also has dual vents on the sides to allow unrestricted inhalation and exhalation. If oxygen flow is set too low for patient requirement, the vents will allow the patient to inhale ambient air. If oxygen flow is set too high for patient requirement, the vents will allow excess air to escape.

Operation:

1. Connect O2 source to mask using the tubing provided or remove the 22mm O2 fitting from the mask to connect to an anesthesia machine breathing circuit.
2. Apply mask to patient's nose and mouth, creating a seal.
3. Set appropriate O2 flow rate for the patient.
 - a. Feline Mask: 1-3 liters O2
 - b. Small Canine Mask: 3-5 liters O2
 - c. Large Canine Mask: 5-7 liters O2

***Note: For long term insufflation, oxygen should flow through a bubble type humidifier to prevent mucosal drying.**

Cleaning Instructions:

The masks and tubing should be cleaned after each use with dilute chlorhexidine solution or mild soap and water. For the best method of cleaning, remove the diaphragm and 22mm adapter from the mask. Then dry parts and reinstall diaphragm and adapter for the next use.

Smiths Medical PM, Inc.

Veterinary Division

N7 W22025 Johnson Drive, Waukesha, WI 53186 USA

Phone: 262-513-8500 Fax: 262-513-9069 Toll-Free USA: 888-745-6562

www.surgivet.com

Smiths Medical - a part of Smiths Group plc

Smiths Medical design mark and SurgiVet are trademarks of the Smiths Medical family of companies.

The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark office and certain other countries.


© 2006 Smiths Medical family of companies. All rights reserved.

Name RecoveryO2Masks Rev. 00 Date 2/22/07

SurgiVet

Clinical Support Services

Toll-Free USA: 888-745-6562

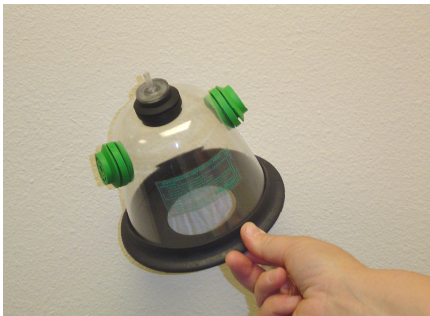

Recovery O2 mask ready to be used with O2 supply tubing that is provided.



Recovery O2 mask ready to be used with a 22mm anesthesia breathing circuit.



Once connected to O2 source, apply mask over the patient's nose and mouth as shown and set O2 source to appropriate flow rate for the patient.

Feline mask: 1-3 liters
Small Canine mask: 3-5 liters
Large Canine mask: 5-7 liters

Smiths Medical PM, Inc.

Veterinary Division

N7 W22025 Johnson Drive, Waukesha, WI 53186 USA

Phone: 262-513-8500 Fax: 262-513-9069 Toll-Free USA: 888-745-6562

www.surgivet.com

Smiths Medical - a part of Smiths Group plc

Smiths Medical design mark and SurgiVet are trademarks of the Smiths Medical family of companies.

The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark office and certain other countries.

© 2006 Smiths Medical family of companies. All rights reserved.

Name RecoveryO2Masks Rev. 00 Date 2/22/07